

Reprocessing & Material Compatibility | CARESTREAM Touch Ultrasound Transducers

Reprocessing Methods for the Carestream Touch Prime Transducer Series

Validated Reprocessing Methods		Transducers														
		6C2	6C2s	9C2	14L3	13L4W	10L2W	18L5	E14C4t	E14C4	E14CL4b	E10C4	20R3	N13C5	5P1	
Manual Cleaning ^a	3E-Zyme	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
	Manual Disinfection ^a															
Manual Disinfection ^a	Korsolex Basic	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
	Ethanol 70% (wiping)	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
	Revita-Ox Resert/Resert XL HLD	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
	Tristel Fuse For Instruments	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
Automated Disinfection ^a																
Sterilization ^a	Medivators Advantage Plus Intercept (detergent), Rapicide PA Disinfectant flush: 70% isopropyl alcohol	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
	STERIS System 1, 1E, 1 Plus and 1 Express ^b	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
	STERIS V-Pro 1, 1 Plus, V-Pro maX Standard cycle	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
	STERIS V-Pro 60 Non Lumen cycle	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
	Sterrad NX & 100NX Standard cycle	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
Sterrad 100S Standard cycle (USA), Short cycle (rest of the world)	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•

Reprocessing & Material Compatibility | CARESTREAM Touch Ultrasound Transducers

Material Compatibility Reprocessing Methods for Carestream Touch Prime Transducer Series

Material Compatible Reprocessing Methods		Transducers														
		6C2	6C2s	9C2	14L3	13L4w	10L2w	18L5	E14C4t	E14C4	E14CL4b	E10C4	20R3	N13C5	5P1	
Cleaning^a	pH neutral (pH 6-8), non-corrosive cleaning products intended for medical devices	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Intercept Wipes	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Intercept Detergent	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	3E-Zyme (Automated use)	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Korsolex Endo-Cleaner 0,5%	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
Disinfection^a	Cavi Wipes	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	CaviCide	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Cidex OPA	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Gigasept FF	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Glutaraldehyde 2% – 3.4%	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Isopropanol 70% (wiping)	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Korsolex Endo Disinfectant 1%	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Korsolex Extra	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Korsolex Basic (Automated use)	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Meliseptol Foam	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Metricide OPA Plus	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Neodisher Endo Sept GA	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Neodisher Septo DN	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Nu-Cidex	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Rapicide	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	RAPICIDE OPA/28	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Rely+On Perasafe	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Revital-Ox Resert / Resert XL HLD (Automated use)	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Sani Cloth Plus Wipes	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Sani Cloth Super Wipes	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
	Sekusept Aktiv	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*
Thermosept PAA	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	
Tristel Duo for Ultrasound	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	
Tristel Trio Wipe System	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	
Trophon EPR	*	*	*	*	*	*	*	*	*	*	*	*	*	*	*	