

# Compatible Cleaning Products for the Touch Prime Transducer Series

The following table indicates products that have been evaluated for chemical and material compatibility with Touch transducers and not for their efficacy in attaining the appropriate level of disinfection. Consult the product manufacturer for information about the biological effectiveness of each solution.

Follow the product manufacturer's instructions. Using a non-recommended solution, an incorrect solution strength, or immersing a transducer longer or deeper than recommended by the product manufacturer can damage the transducer. Ensure that the reprocessing chamber is an appropriate size for the transducer.

<b>Product</b>	<b>Manufacturer</b>
3E-ZYME (automated use)	Medisafe UK Limited
ACCEL PREVENTION	VIROX Technologies, Inc.
ADASPOR Single Shot	CANTEL MEDICA (Italy) S.R.L.
BOMIX Plus BODE	Chemie GmbH
CAVIWIPES	Metrex Research
CAVICIDE	Metrex Research
CIDEX OPA	Advanced Sterilization Products (ASP)
CLEANISEPT Wipes Forte	DR. SCHUMACHER GmbH
DISTEL WIPES	TRISTEL SOLUTIONS Limited
Ethanol 70 %	none specific
GIGASEPT FF	Schülke & Mayr GmbH
GIGAZYME	Schülke & Mayr GmbH
Glutaraldehyde 2–3.4 %	none specific
INTERCEPT Wipes	MEDIVATORS Inc.
Isopropanol 70 % (wiping)	MEDIVATORS Inc.
KORSOLEX Basic (automated use)	BODE Chemie GmbH
KORSOLEX Endo-Cleaner 0.5 %	BODE Chemie GmbH
KORSOLEX Endo-Disinfectant 1 %	BODE Chemie GmbH
KORSOLEX Extra	BODE Chemie GmbH
MELISEPTOL Foam	B. Braun
METRICIDE OPA Plus	Metrex

## Reprocessing & Material Compatibility | CARESTREAM Touch Ultrasound Transducers

Product	Manufacturer
MIKROBAC Tissues	BODE Chemie GmbH
NEODISHER Endo SEPT GA	Chemische Fabrik Dr. Weigert GmbH & Co. KG
NEODISHER Endo SEPT PAC	Chemische Fabrik Dr. Weigert GmbH & Co. KG
NEODISHER MediClean Forte	Chemische Fabrik Dr. Weigert GmbH & Co. KG
NEODISHER Septo DN	Chemische Fabrik Dr. Weigert GmbH & Co. KG
NEODISHER Septo PAC	Chemische Fabrik Dr. Weigert GmbH & Co. KG
NU-CIDEX	Advanced Sterilization Products (ASP)
OPAL	Whiteley Medical
RAPCIDE	MEDIVATORS Inc.
RAPCIDE OPA/28	MEDIVATORS Inc.
RAPCIDE PA Disinfectant	MEDIVATORS Inc.
RELY+ON PERASAFE	DuPont
REVITAL-OX RESERT, RESERT XL HLD (automated use)	STERIS Corporation
SANI-CLOTH Plus Wipes	PDI
SANI-CLOTH Super Wipes	PDI
SEKUSEPT Aktiv	Ecolab A/S
SEKUSEPT MultiEnzyme P	Ecolab Deutschland GmbH
STERIS System 1/STERIS System 1E	STERIS Corporation
STERIS V-PRO 1/STERIS V-PRO 1 Plus/STERIS V-PRO 60/STERIS V-PRO maX	STERIS Corporation
STERRAD 100S/ 200/ STERRAD NX/ STERRAD 100NX	Advanced Sterilization Products (ASP)
THERMOSEPT PAA	Schülke & Mayr GmbH
TRISTEL Duo for Ultrasound	Tristel Solutions Limited
TRISTEL Fuse for Instruments	Tristel Solutions Limited
TRISTEL Trio Wipe System	Tristel Solutions Limited
TROPHON EPR	Nanosonics Ltd
Wet Wipe Ethanol Disinfection	Wet Wipe A/S