

Carestream's Dental Imaging Software Installation Guide



Important: U.S. users see the proper installation guide delivered in the Carestream's Dental Imaging Software box.

Before installing the Carestream's Dental Imaging Software, check that:

- The computer has all the PC system requirements
- You have the software DVD as of **version 6.12 or higher**

To install the Carestream's Dental Imaging Software, follow these steps:

- 1 Insert the software DVD in the DVD-ROM drive of the computer.

Wait for the installation program to start. If the program does not start automatically, click **Start > Run** and enter **D:\setup.exe** if D is the letter for the DVD-ROM drive, or the letter of the relevant drive on the computer.


- 2 The **Choose Setup Language** dialog box is displayed. Select the installation language and click **OK**.




- 3 The **Dental Imaging Software** welcome page and the **InstallShield wizard** are displayed.




- 4 The **Welcome to Dental Imaging Software Installation** dialog box is displayed. Click **Next** to launch the installation.

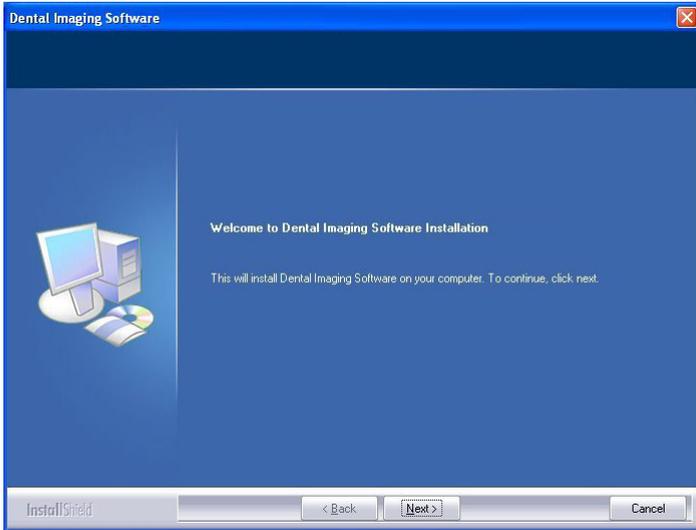

- 5 The **License Agreement** dialog box is displayed. Accept and click **Yes**.

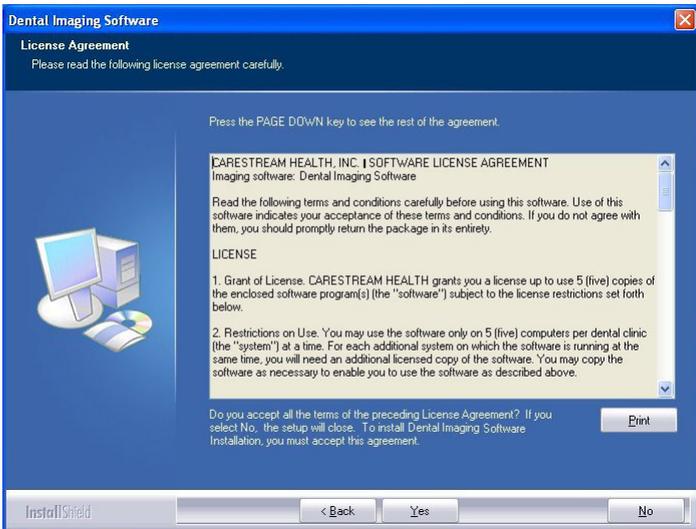


- 6 The **Choose Destination Location** dialog box is displayed. Click **Next** if you accept the default destination folder (c:\program files\Kodak\Kodak Dental Imaging Software) or browse to choose another destination folder.

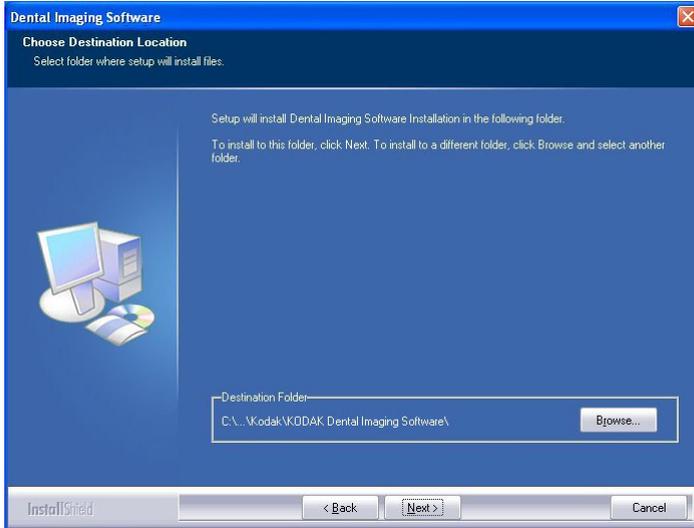


- 7 The **Dental Imaging Software** dialog box is displayed. The **Patient file** is selected by default but you must select the device.

8 To select the desired device, do the following:

- Click on the drop-down list of **No Intraoral Camera Installation**.
- Select **Intraoral Camera Installation**.

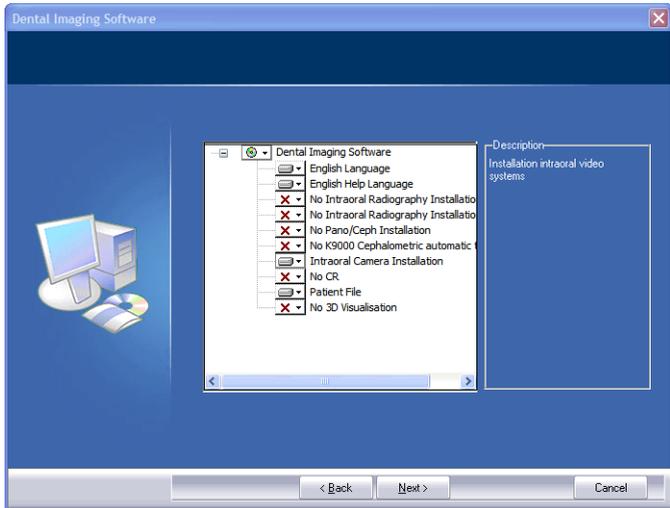


- Click **Next**.
- Check the **CS1600/CS1500/CS1200 camera** check box.

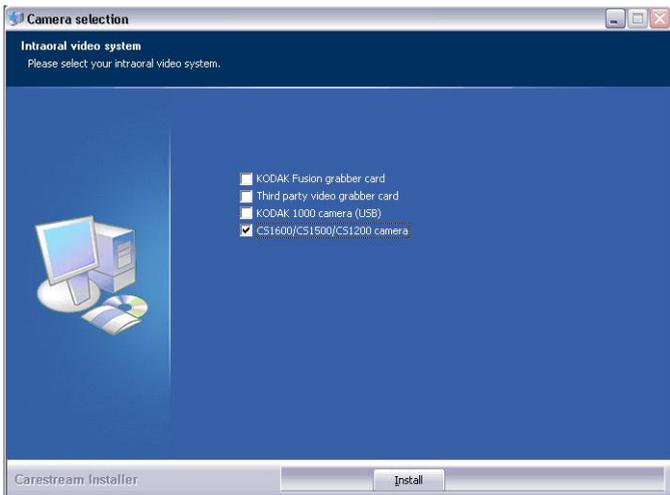

- Click **Install** to begin the installation.

The **InstallShield Wizard** is displayed. Let the installation process run automatically.

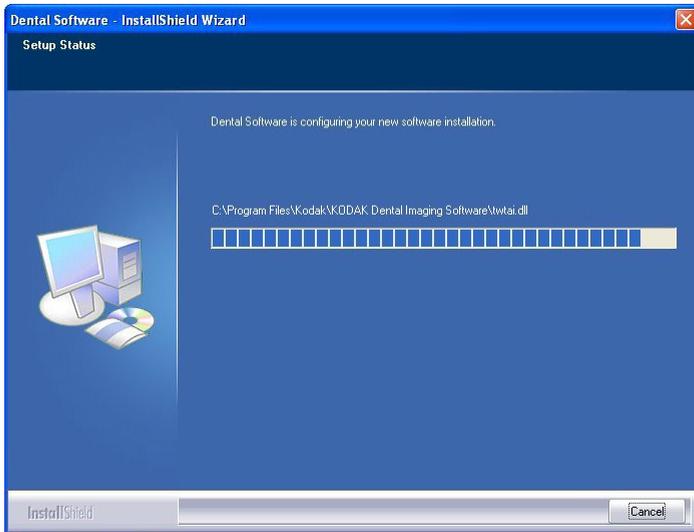

The **Patient file - InstallShield Wizard** is displayed while the patient files are installed on the hard drive.




The **IO Camera InstallShield Wizard** launches the installation.

Before the installation complete, the **Software Installation** alert dialog box is displayed. Click **Continue Anyway**.



- 9 The **Installation Complete** dialog box is displayed when the installation is finished. Click **Finish**.

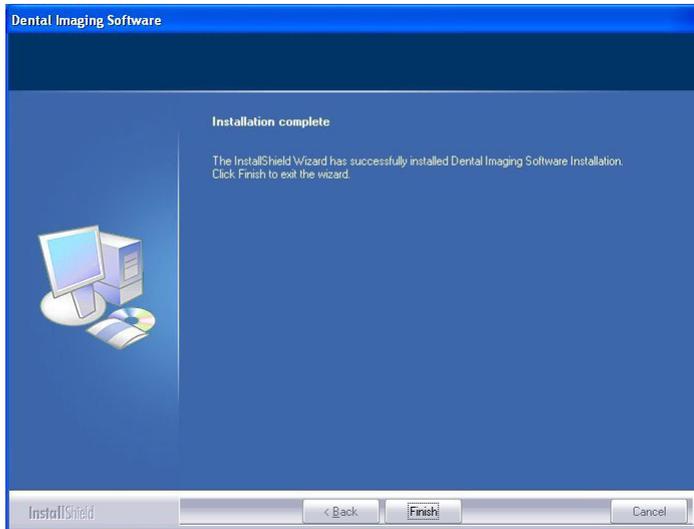

The  is installed on your desktop.



Note: When opening an **Acquisition** interface for the first time, you may see the following **Windows Security Alert**. If so, click **Unblock**.

